

# Health Care Call System

## Pullcord Stations Series 2300

### Description

The Jeron Cat. 2320, 2322, 2331 and 2332 pullcord stations are used in the EC-300 Tone-Visual Nurse Call Systems to activate call signals from a room, bed or bath.

- Pullcord Activation of Bath Calls
- Optional Call Placed Indicator

### Operation

Pulling the cord activates all associated system call signals: dome and zone lamps illuminate, corresponding room and call placed indicator illuminate at the master annunciator, accompanied by an audible alert tone. If present, optional call placed indicator illuminates. The calls must be canceled by reset switch at the originating pull station.

Stations may be used in single, dual, or triple status systems. Call distinction depends on system configuration.

### Equipment

The stations are equipped with a pullcord, reset switch and engraved PULL FOR HELP label on a stainless steel trimplate for flush wall mount. Optional call placed indicator is available. The callcords are replaceable and the panels withstand common hospital cleaning agents.



### Architects/ Engineers Specifications

The stations shall be the Jeron Series 2300 with six foot white nylon pullcord, plastic pendant, reset button and engraved PULL FOR HELP label on a stainless steel face plate for flush wall mount. If specified, stations shall be equipped with optional call placed indicator.

### Technical Data

Mounting:

Brushed stainless steel 1-gang face-plate with stainless steel mounting hardware

Dimensions (H x W x D):

4.5 x 2.75 x 1.75 in.  
 (11.4 x 7 x 4.5 cm)

Backbox Required:

Raco 670RAC 1-gang or equivalent

Controls/Indicators:

Red LED call placed indicator (optional)

6 ft. (1.8 m) braided white nylon cord with matching plastic pendant

Slide action reset switch

Operating Voltage: 28 VDC

Field Connections:

8 in. (20 cm) color coded wire leads with stripped ends and wire nuts

Certification:

UL 1069 Listed  
 C-UL—Canadian Standard  
 C22.2 NO.205

### Associated Jeron Equipment

- Cat. 23002 Call Placed Indicator option

Cat. #	Contacts	Single Status System		Dual or Triple Status System	
		room, bed bath	Zoned	room, bed, stand-alone, bath	Zoned
2320	SPST				
2322	DPST		room, bed bath		room, bed stand-alone bath
2331	SPDT			bath associated with a room	
2332	DPDT				bath associated with a room

PRODUCT IS SUBJECT TO CHANGE WITHOUT NOTIFICATION